

Microgen® Listeria-ID

An identification system for *Listeria* spp.

- A complete identification system, no additional materials required
- Substrates included conform to all international standards
- Enhanced built in haemolysis test—CAMP test not required
- Validated for chromogenic media
- AFNOR validated with BioMerieux ALOA medium
- Inoculated directly from selective media (no need for sub-culturing before inoculation)
- AOAC—RI validated
- CMMAS Assessed and Approved

Microgen® Listeria-ID employs 12 standardised microwell substrates combined with the Microgen Identification System Software to identify members of the genus *Listeria*.

AOAC-RI validated
Easy to use
No additional materials
required



- *Listeria monocytogenes*
- *Listeria welshimeri*
- *Listeria ivanovii*
- *Listeria innocua*
- *Listeria grayi*
- *Listeria seeligeri*

The above organisms can be identified directly from selective agar or non-selective agar using Microgen Listeria-ID. Identification is achieved using all the tests recommended in international standard methods for the identification of *Listeria* spp. without the need for additional confirmatory tests.

Microgen® Listeria-ID

Microgen® Listeria-ID has optimised the differentiation of haemolytic and non-haemolytic *Listeria* spp. by improving the readability of the micro-haemolysis reaction, which eliminates the need to sub-culture from selective agar or perform a CAMP test. The readability of a negative haemolysis reaction has been improved; if an organism does not produce haemolysin, the red blood cells will remain intact and form a distinct red pellet. If an organism does produce haemolysin, the red cells will rapidly

be lysed and cellular contents released as a red/brown solution, indicating a positive result.

As with all Microgen biochemical ID systems, Listeria-ID is supported by user friendly Computer Identification System Software (MID60), delivering the most comprehensive and up-to-date databases which registered users can update free-of-charge from the Microgen® Bioproducts website.

Ordering Information:

Code	Description	Size	Storage
MID-67	Microgen® Listeria-ID	20 tests	2 - 8°C

If you need any further information on any of the above tests please contact Microgen Bioproducts Ltd.

MICROGEN
BIOPRODUCTS

Microgen Bioproducts Ltd, Unit 1, Watchmoor Point, Camberley, Surrey GU15 3AD, United Kingdom
T: +44 (0)1276 600 081 F: +44 (0)1276 600 151 E: sales.camberley@lab21.com
www.microgenbioproducts.com